

QUICR

Quantum
Implant Cementer

Usage Instructions




Take **Part B** and screw into **Part A** so it is fully seated.



Place the Tibase/Abutment on the assembled **Part A/B**.


2A. Treat Tibase/Abutment And Crown.

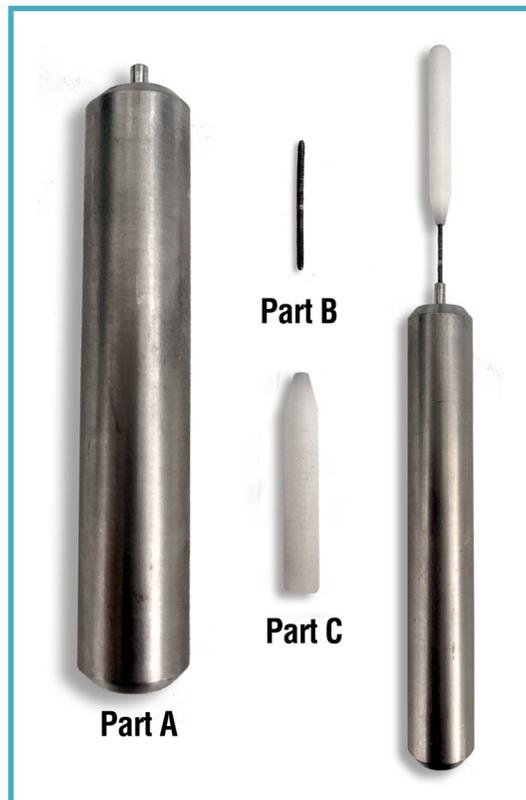

Mix cement and place on Tibase/Abutment.



Place crown on Tibase/Abutment.

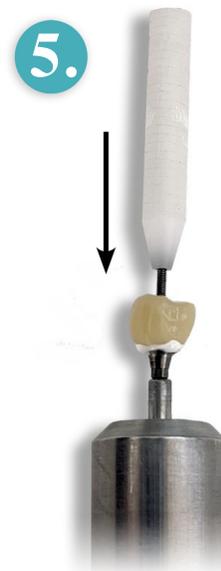

Place **Part C** onto **Part A/B** and screw down onto crown until crown is held snugly in place.



Clean off excess cement and visually inspect the seal to verify it is perfect.

Then set it down and walk away.